



EZ Way, Inc.

EZ Way Classic Stand Competency Quiz

Questions

Question Group 1

1. The battery has been discharged and needs to be charged or replaced when?

(1 point)

- (A) The control box beeps when the up or down button is pressed
- (B) The LED on the control box turns red
- (C) The LED on the hand control turns yellow when the up or down button is pressed
- (D) A & C

2. The hand control is the only method to raise/lower the resident/patient under normal operation?

(1 point)

- True
- False

3. Pressing the emergency stop button will immediately stop the machine from moving.

(1 point)

- True
- False

4. How do you manually lower a patient if the machine stops working?

(1 point)

- Use scissors to cut the sling straps
- Turn the red collar on the shaft of the actuator clockwise
- Push the emergency stop button
- Use the hand control

5. If the harness shows signs of tearing or excessive wear it is okay to use just once more.

(1 point)

- True
- False



EZ Way, Inc.

EZ Way Classic Stand Competency Quiz

6. You should lock the wheels on the stand when raising a patient prior to ambulation with the footplate removed.

(1 point)

- True
- False

7. The resident/patient does not need to bear some weight when being transferred with the Classic Stand.

(1 point)

- True
- False

8. Where should the harness be positioned on the resident/patient:

(1 point)

- Under the buttocks
- Around the torso and outside of the arms
- Around the shoulders
- Around the torso (2" to 3" below underarms) and inside of the arms

9. You should attach the harness to the stand with the stand arms in the highest position.

(1 point)

- True
- False

10. You should tighten the strap on the harness as you raise the resident/patient.

(1 point)

- True
- False



EZ Way, Inc.

EZ Way Classic Stand Competency Quiz

11. Which of the following is a use for the straps on the back of the sling?

(Multiple Choice Question, 1 point)

- Decoration
- To attach the sling to the lift
- To position the patient in a chair
- All of the above

12. You should always use the longest loops of the sling.

(True/False Question, 1 point)

- True
- False